COOPER

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AUDIBABY Notice 297 X 210 mm

Pays : International

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AUDISPRAY.COM

300492
AU7665_V01

Date of last revision of the instructions for use: Mai 2021.

EN AUDIBABY

AUDIBABY is a hypertonic solution, of microbiologically controlled seawater. Its ion content and trace elements makes this solution effective for cleaning the ears of infants and young children without discomfort.

What is AUDIBABY ?

AUDIBABY keeps the ears of the baby and young child clean and keeps the ear canal free from impurities (cerumen, desquamation, hair, etc.) by facilitating their natural evacuation. AUDIBABY facilitates the examination of tympanum by the doctor. A clean ear helps maintain a good hearing. Used regularly, AUDIBABY prevents the accumulation of cerumen. AUDIBABY can be used from birth without affecting ear sensitivity. Presented in single use disposable doses, AUDIBABY allows a regular and practical hygiene of the ear.

Who can use AUDIBABY ?

From 0 to 12 month: 6 - 10 drops in each ear, once a week
From 12 to 36 months: half of 1 single-dose in each ear, once a week

How to use AUDIBABY ?

1. Detach a single dose.
2. Keep the single-dose for a few seconds in the hand to warm the solution and make it more pleasant for the child.
3. Lay the baby on its side. Open the single dose and place it at the opening of the ear. Exert gentle pressure on the single dose to pour a few drops.
4. For maximum efficiency, massage the base of the ear for 10 seconds, before wiping the excess of liquid.
5. Repeat the operation for the 2nd ear and dispose of the single dose.

Contraindications

Do not use in case of otitis or generally in any child with a medical history or surgical procedures in the ear or eardrum.

Precaution for use

In case of doubt regarding the use of the product, ask your pharmacist or doctor for advice. Do not exceed the expiration date indicated on packaging. Keep out of the reach of children.

Any serious incident in connection with AUDIBABY should be notify to the manufacturer and to the competent authority of the state in which the user is established. For single use only. As the product is preservative-free, do not reuse an opened single dose.

FOR CHILDREN UP FROM 3 YEARS USE AUDISPRAY JUNIOR FOR A REGULAR EAR CARE

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